

Cannulaide

Nasal Seal for
Infant Nasal CPAP



Supplied by:
SLE

When the smallest thing matters

A good seal from the first breath

- Aids in continuous, progressive recruitment of alveoli.
- Helps prevent damage to nasal septum and nasal epithelium.
- Helps reduce “oversizing” of prongs due to nasal dilation.
- Provides a positive seal the first time, every time.
- Secures prongs in place with soft Velcro.
- Maintains seal during patient movements.
- Keeps cannula barrel away from nasal septum.
- Fits popular nasal prongs.
- 5 sizes to fit infants from 400 g to 4 kg.
- Pressure seal between prongs and Cannulaide, not between prongs and baby’s skin.
- Box of 25.

How to Apply

1. Remove from package



2. Smooth over patient’s nose



3. Don’t forget to apply nasal prongs

Ordering Details

Part N°	Size	Description
N4410/100	0	For babies < 700 g
N4410/101	1	For babies < 1250 g
N4410/102	2	For babies < 2000 g
N4410/103	3	For babies < 3000 g
N4410/104	4	For babies > 3000 g

Boxes of 25 pcs



Supplied by:
SLE
SLE Limited.
Twin Bridges Business Park, 232 Selsdon Road,
South Croydon Surrey CR2 6PL UK
Telephone: +44 (0)20 8681 1414 • Fax: +44 (0)20 8649 8570
E-mail: sales@sle.co.uk • Web: www.sle.co.uk



Dealing with nasal septal breakdown?

Before you switch nasal prongs, try Cannulaide.

Many hospitals have switched from one brand of nasal prong to another looking for a way to reduce nasal septal breakdown. Some have found an improvement while others have not. In our experience this seems to be independent of the brand of nasal prong. Each one has its strengths and weaknesses.

Keys to success in nCPAP

1. Maintain the pressure seal.
2. Keep the prongs from irritating the skin.

Cannulaide addresses both.

Sized to fit the prongs

You pick the nasal prongs to fit the baby’s nares. Without the Cannulaide, the prongs can dilate the nares within hours, forcing you to switch to a larger size prong in order to maintain a pressure seal. With the Cannulaide, the prongs don’t have to touch the skin in order to make a good seal. Less irritation *plus* a better, more consistent, and longer lasting seal with the proper size prongs all along. A consistent seal aids in the continuous, progressive recruitment of alveoli.

Fixes the prongs in place

Cannulaide’s gentle hydrocolloid adhesive plus low profile, soft Velcro holds the prongs in place, yet allows you to remove the prongs for inspection. It also keeps the cannula barrel away from the septum, plus it reduces the application and removal of adhesives. (The Velcro can also be used to secure an OG tube, eliminating yet another adhesive.)

Manufactured by:

Beevers Mfg & Supply
(503) 472-9055

CSDD0008 01/12