

NEO-fit

Neonatal
Endotracheal Tube Grip



Supplied by:
SLE


When the smallest thing matters

Features

Provides rapid, secure stabilisation and allows changes in tube position without removing tape from tube or patient.

One size fits 2.5 mm to 4.0 mm tubes.

The unique Velcro® strap provides rapid and secure grip on to the endotracheal tube and fast release and refit, making tube adjustment simple.

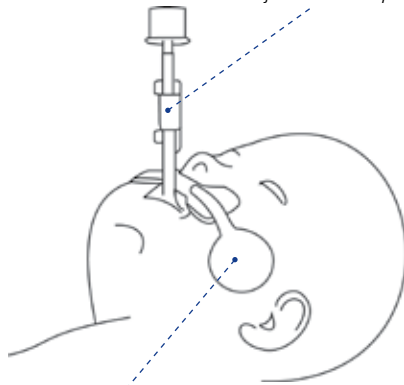

Bifurcated flex-pads provide secure facial contact.

Order details

N4225/40

Each kit includes one plastic tube grip with adjustable Velcro® strap. Two adhesive-backed foam pads, two 50 mm foam 'lollipop' securing tapes (optional use).

Box of 20 or box of 80 (please specify)

Application Guide

1. Trial fit the NEO-fit on patient prior to removing the adhesive liner, (adhesive pad should not overlap the mouth or eyes). Usage of a pectin-based skin preparation may minimise the risk of contact dermatitis

2. Remove the adhesive liners from the bottom of the adhesive pads using the pull tab provided. (See figure 2) Caution: Do not peel the adhesive pad away from the plastic base during removal of the adhesive liner or the adhesive may be adversely affected. Use a new NEO-fit if this occurs.




3. Affix the adhesive pads to the patient adjacent to the ET tube. (See figure 3).

4. Wrap the tube holder strap snugly around the ET tube by pulling from the end of the strap. Pushing the strap around the tube may not secure the tube adequately. Do not wrap too tightly as this may interfere with the passage of a suction catheter through the ET tube. (See figure 4).

5. Secure the strap using the hook and loop material of the strap. If needed, secure using "lollipop" foam adhesive tape included, or other tape as appropriate. (See figure 5).

Warnings:

- Separation of the plastic base foot from the adhesive foam pad may result in potential ET tube extubation. If signs of delamination are observed, replace with a new NEO-fit.
- Usage of ointments or oils may adversely affect the adhesion properties of the adhesive foam pad.
- Excessive salivation may adversely affect the attachment of this device or cause the device to fail during use.
- Discontinue use immediately if skin irritation occurs.

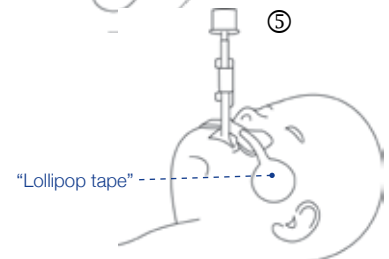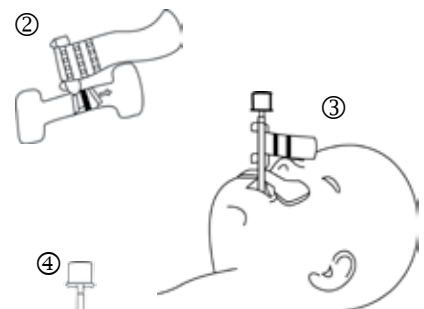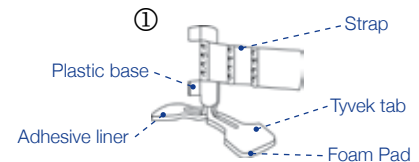

Precautions:

- For stabilising a 2.5 mm to 4.0 mm ID neonatal ET tube. Not for use with 2.0 mm ID or smaller ET tube.
- Do not use as sole method of support for ventilator tubes: use a tubing support device.
- Patient must be under professional supervision while intubated.
- Do not wrap the tube holder too tightly or the ET tube may be deformed and prevent suction catheter passage.
- Do not wrap the tube holder too loosely or the ET tube may move in the tube holder.