

Sticky Whiskers

Gentle security for
nasal cannulas



Supplied by:
SLE

When the smallest thing matters

Features

- ✓ Attaches with gentle, non-irritating hydrocolloid adhesive
- ✓ Secures cannula in place with soft, low-profile Velcro®
- ✓ Allows for easy positioning of cannula
- ✓ One size fits all
- ✓ Can be used for securing an OG

Keep the “O”s in the Nose

Securing tubes to infants has proven to be a challenge. In the past, cannulas have often been held in place with tapes that are too aggressive, causing skin issues or requiring significant handling of the infant in order to relocate the device. However, if the adhesive is too light, the tubes become dislodged.

The Sticky Whiskers Velcro® interface allows a cannula to be secure yet still repositionable.


After applying the Sticky Whiskers™ to the upper lip, simply wrap the soft Velcro® loop strips around the cannula “like a candy cane” and place your cannula in place directly under the nose. No tension on the septum, ears or taping the tubing to the face causing face creases.

The Sticky Whiskers™ can also be used to secure OG tubes if necessary. The soft Velcro® tabs can be secured to the OG tube in a “flag” configuration. When it’s time to change sides, the flag can be moved from one side to the other with good confidence of tube depth being consistent.


Ordering Details

Part No. Description

N4406 Sticky Whiskers (25 pcs)

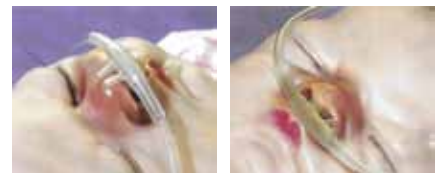

Avoid nasal trauma

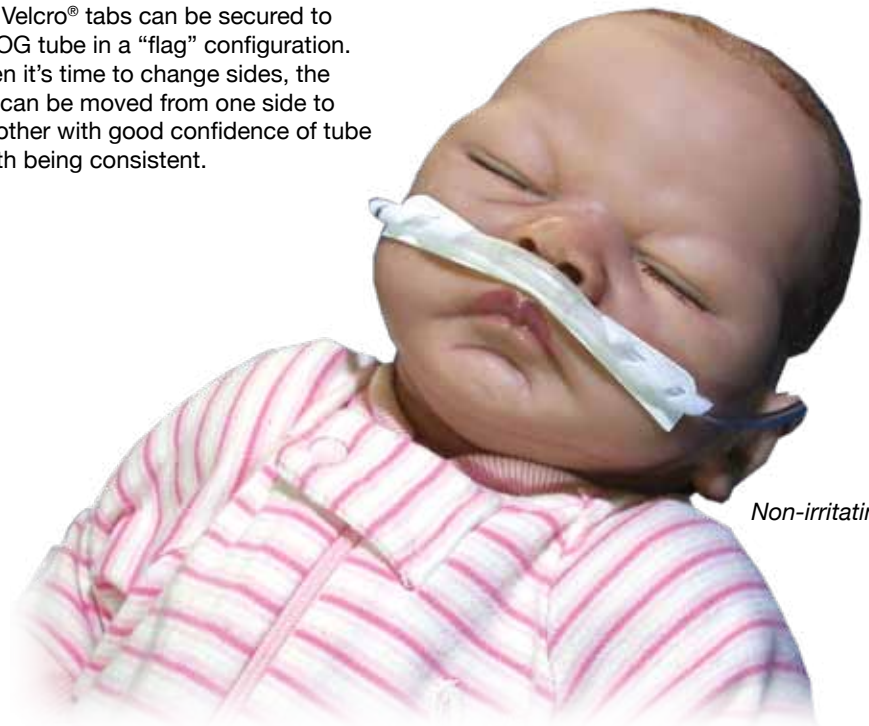

Non-irritating



Supplied by:

SLE

SLE Limited.
Twin Bridges Business Park, 232 Selsdon Road,
South Croydon Surrey CR2 6PL UK
Telephone: +44 (0)20 8681 1414 • Fax: +44 (0)20 8649 8570
E-mail: sales@sle.co.uk • Web: www.sle.co.uk

Manufactured by:
Beavers Mfg & Supply, Inc
14670 Baker Creek Rd
McMinnville Oregon 97128
US

OSD0019 01/12